


Cleaning the Corda miniFit (thin tube)




The following pictures illustrate how to clean a hearing aid with a thin tube.



1. Detach the corda tube by pulling the tube away from the hearing instrument.

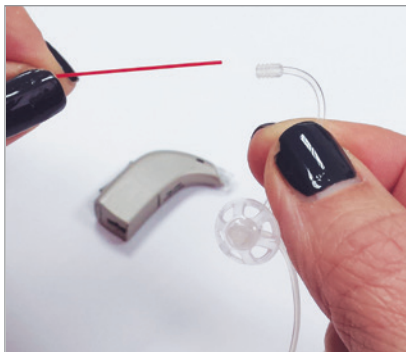


2. Insert the cleaning tool into the top part of the tube and feed it all the way through to the dome.

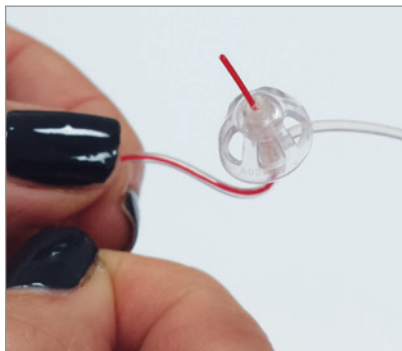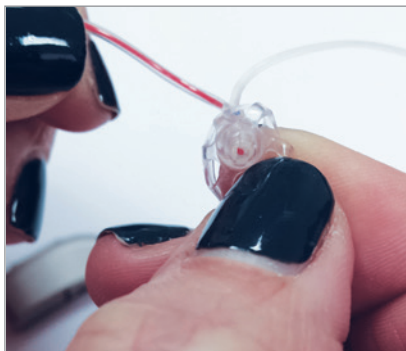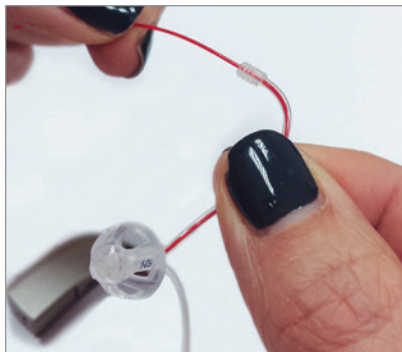

3. Carefully push the cleaning tool through the dome past the vent, without removing the dome.

PLEASE NOTE:

1. If the dome is removed, always replace with a **NEW** dome.
2. Never re-attach a used dome back onto the tube.
3. If the dome is missing **DO NOT USE**. Please call audiology.
4. Always change the dome & tube as one unit - do not change the dome on its own.
5. If the dome is pushed too far and it comes through the wax guard please replace the dome.

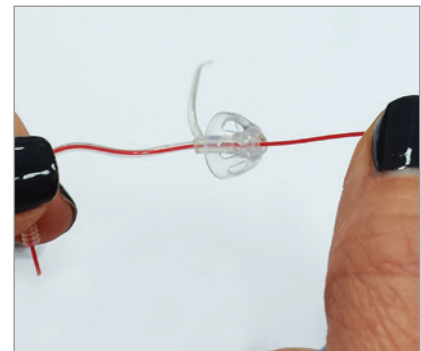

Always clean the dome from tip to base



4. To clean the dome, wipe the dome gently with a wet-wipe or damp cloth to remove any excess cerumen (wax).




5. Re-attach the corda tube by pushing it back on to the hearing instrument with a simple click fit action.



6. Correctly attached tube, dome and hearing instrument.



Place your finger on the bend of the tube and gently push to place the dome in the correct position. Do not try to push the dome itself.

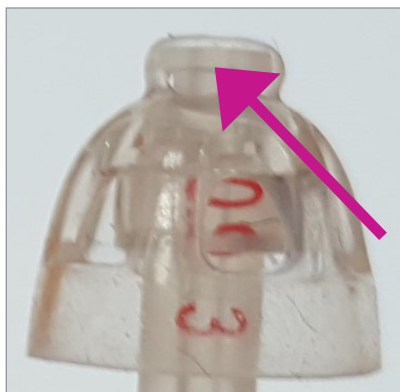

If the dome is pushed too far it can come through the wax guard (left image).

If this happens replace dome and tube and do not use until this is done.

Correct image of dome (right image).

